



IVTAG Device Instructions for Use

Application

1. Remove IVTAG from storage container.
2. Use ball point pen or ultrafine sharpie to write relevant information on IVTAG designated areas.
3. Let Ink Dry
4. Wrap extended tab around desired tubing and press snaps together, securing IVTAG to tubing.
5. IVTAGs is usable for all types of Central Vascular Access Device tubing.


- Non-Sterile Product unless otherwise indicated in alternate packaging.

- Keep out of reach of children. In case of accidental swallowing or choking, seek professional assistance.

Model No: IVTAG1-325916

Patent pending

Made in the USA.

IVTAGS, LLC 

www.IVTAGS.com

IVTAGS and IVTAGS.com are registered trademarks of IVTAGS, LLC.



Do not reuse



No Latex



Do not use if
package is damaged